



Print your medical imaging in HD colour with integrated DICOM A4 printers

OKI is a leader in printing innovation, providing organisations with High Definition colour printing devices based on digital LED technology, and now our sophisticated range of printers integrate seamlessly with your clinical systems using DICOM.

A first in medical imaging and printing

This is a first in digital printer technology, combining the cost effectiveness and high quality output of an LED printer with embedded DICOM software, allowing you to print directly from medical equipment without the use of conversion software or external print servers. You no longer need to settle for expensive printing on specialist media and devices.

With the C610 DM you can now print your medical imaging cost effectively on standard media formats in vibrant, HD colour and in sizes up to A4. From nuclear medicine (2D Scintigraphy and 3D SPECT tomography) to ultrasounds, these images can now be easily printed for your own non-diagnostic use or to provide a copy for your patients and clients to take away with them.

Reduce your costs and improve your printing

The C610 DM is not only perfect for your medical imaging printing, it can also be used for your standard office printing, from patient letters and forms to medical reports, allowing you to choose one machine that answers all the needs of your organisation, cutting down on the number of devices and consumables used.

The C610 DM is based on our existing, award winning printers, offering the same reliability, ease of use, low total cost of ownership, media flexibility and high quality output, with the additional benefit of embedded DICOM technology. Through system integration, the use of energy efficient equipment, standard printer consumables and a three year on-site warranty, the C610 DM offers considerable savings for your day-to-day printing needs.

See it clearer with OKI

Talking directly with your medical imaging equipment, the C610 DM presents many opportunities for printing high quality, non-diagnostic images in HD mono and colour on a wide range of media. The significantly reduced costs of printing using the C610 DM, compared with traditional DICOM printing technologies, allows medical establishments to change their approach to printing.

Imagine the possibilities; offer mothers-to-be A4 ultrasound scans or keep sharp and clear non-diagnostic images in patients' records, and all at a very low cost. In fact, the same cost of printing as standard office documents.

Benefits at a Glance

- Fully integrated DICOM print server
- Up to 2 simultaneous imaging equipment connections
- Recommended usage - Nuclear medicine, ultrasound
- High quality non-diagnostic mono and colour printing of medical images
- No additional external hardware or software required
- Handles media from A6 to A4 wide 1200mm banners, from 64 to 250gsm
- Three year on-site warranty (product to be registered within 30 days of purchase)



C610 DM - Colour and Mono DICOM Embedded Printer

OVERVIEW

Description	The C610 DM is our most compact DICOM imager. Primarily designed for Ultrasonography, it can of course be connected to other DICOM print SCU compliant modalities
Imaging Technology	CMYK toner based engine with tandem technology. Light sources are LED arrays @ 600 ppi with multi level capability
Network	10/100 Base TX (RJ45) interface

GENERAL FEATURES

Print Format - Paper weight	A4 - 64 to 250 gsm
Dimensions (HxWxD)	340 x 435 x 546mm
Weight	26kg

DICOM SERVER FEATURES

Capability	DICOM 3.0 Print SCP
Maximum number of AET	25
Max # calling modalities	No limitations
Maximum connections	2 simultaneous
Dicom Server interface	Integrated web server
Log	Extended log capability in case of troubleshooting
LUT support	identity, reverse, lin-od
Image generation	Specific color rendering optimized for medical images
Graphical Header	Capability to add a graphical header on each printout

AET DEPENDENT FEATURES

AET Name	Print SCP Application Entity Title (AET)
Printer Tray	Use preferred paper for your medical prints
Copy	Send one, print what you need
Border Density	Value from Black, White or Client
Empty image Density	Value from Black, White or Client
Replace Header by Logo	Use a graphical header instead of text for enhanced communication on the printouts
Header	Enter text that will print on top of page or select Logo
Footer	Edit the text on page bottom
Contrast	Add or remove contrast on the printouts
Brightness	Add or remove brightness on the printouts
Hi Quality Glossy Black	Print your greyscale images using best composite rendering
Cyan, Magenta and Yellow Gamma	Range from 0.01 to 2.90 by step of 0.01. Values lower than 1.00 will lighten, values higher than 1.00 will darken the image
Smoothing	Prompts C610 to use or not adjustable Sharpen filter to print
Polarity	Force image polarity inversion if needed

ACCESSORIES (ORDER NUMBERS)

2nd paper tray	44274502
3rd paper tray	44274502
Duplex	01272601
Cabinet	01219302

*Toner: A4 pages yield compliant with ISO/IEC 19798. Printer shipped with 2,000 A4 pages.
Medical printing strongly alter these values.

**Drum: A4 pages average life.

Consumables Information: For the protection of your printer, and in order to ensure that you benefit from its full functionality, this model has been designed to operate only with genuine OKI toner cartridges. These can be identified by the OKI trademark. Any other toner cartridge may not operate at all, even if it is described as "compatible", and if it does work, your printer's performance and print quality may be degraded.



3 Year On-site Warranty

Our devices are manufactured to the highest standards of quality and technology, which has been confirmed by independent tests. We are so convinced of the high quality of our products that we are offering you an extension of the standard warranty period to 3 years at no extra cost to you. Simply register your product within 30 days of purchase to be eligible for our unique all inclusive offer. For further information please visit www.oki.co.uk/WARRANTY. Without registration, the standard pan-European 1 year warranty applies.

CONSUMABLES (ORDER NUMBERS)

Toner cartridges* (6,000 pages)	Cyan: 44315307; Magenta: 44315306; Yellow: 44315305
Toner cartridges* (8,000 pages)	Black: 44315308
Image drum** (20,000 pages)	Cyan: 44315107; Magenta: 44315106; Yellow: 44315105; Black: 44315108
Transfer Belt (60,000 pages)	44341902
Fuser Unit (60,000 pages)	44289103

OFFICE USAGE FEATURES

C610 DM inherits all features of standard product	Please refer to C610 specsheet available on OKI websites
--	--

DICOM INTERFACE

On embedded web server	Full configuration of the server features, log viewing
-------------------------------	--

Print SCP AET Configuration

AET Name

OKI

Activate Max Dicom Log

☐

Use US format paper

☐

Printer Tray

AUTO

Copy

CLIENT

Border Density

CLIENT

Empty Image Density

CLIENT

Replace Header by Logo

☐

Header

Using Logo as Header reduce image printing area

Footer

DICOM Embedded Printer - Not for diagnostic use - C

Contrast

0

Brightness

0

Hi Quality Glossy Black Mode

☒

Cyan gamma (0.01 to 2.90)

0.85

Magenta gamma (0.01 to 2.90)

0.89

Yellow gamma (0.01 to 2.90)

0.82

Smoothing Filter

CLIENT

Smoothing Level

LOW

Polarity

CLIENT




OKI EUROPE LTD.

Blays House, Wick Road
Egham, TW20 0HJ
United Kingdom
T +44(0) 20 8219 2190
F +44(0) 20 8219 2199

www.okieurope.com

OKI

Open up your dreams

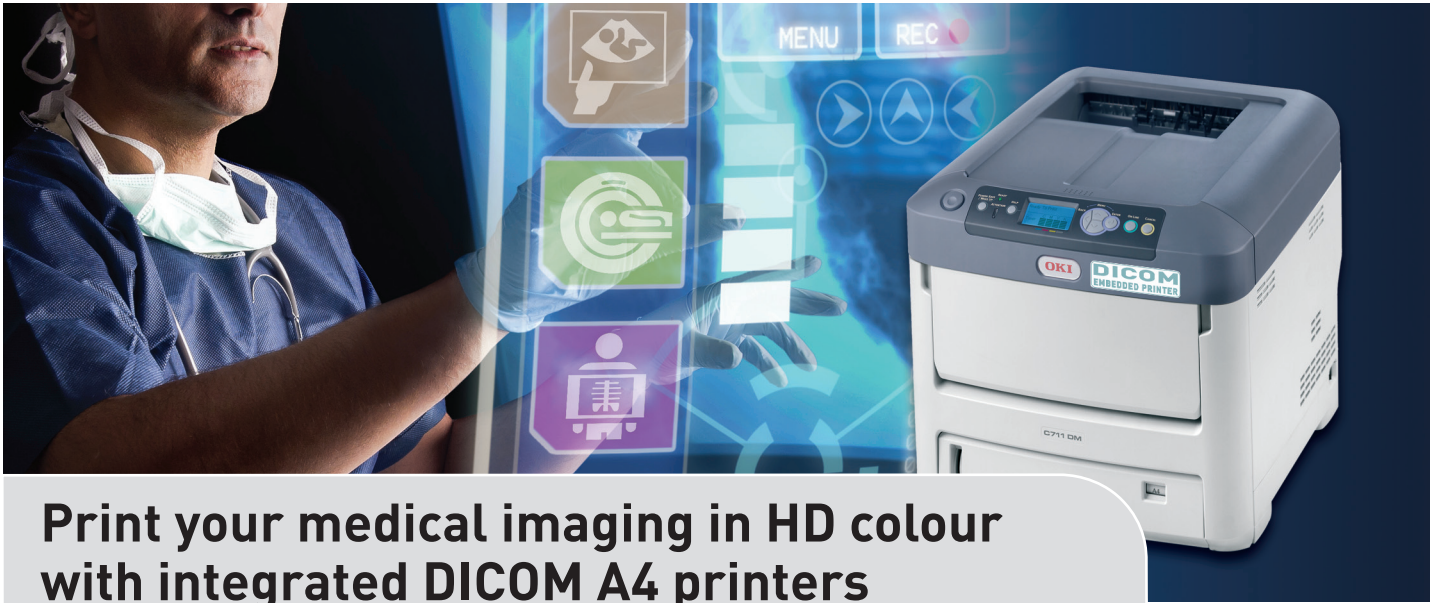

Print your medical imaging in HD colour with integrated DICOM A4 printers

OKI is a leader in printing innovation, providing organisations with High Definition colour printing devices based on digital LED technology, and now our sophisticated range of printers integrate seamlessly with your clinical systems using DICOM.

A first in medical imaging and printing

This is a first in digital printer technology, combining the cost effectiveness and high quality output of an LED printer with embedded DICOM software, allowing you to print directly from medical equipment without the use of conversion software or external print servers. You no longer need to settle for expensive printing on specialist media and devices.

With the C711 DM you can now print your medical imaging cost effectively on standard media formats in vibrant, HD colour and in sizes up to A4. From x-rays (eg dental, veterinary, hospital, clinic) to nuclear medicine, scans (eg MRI, CT) and ultrasounds, these images can now be easily printed for your own non-diagnostic use or to provide a copy for your patients and clients to take away with them.

Reduce your costs and improve your printing

The C711 DM is not only perfect for your medical imaging printing, it can also be used for your standard office printing, from patient letters and forms to medical reports, allowing you to choose one machine that answers all the needs of your organisation, cutting down on the number of devices and consumables used.

The C711 DM is based on our existing, award winning printers, offering the same reliability, ease of use, low total cost of ownership, media flexibility and high quality output, with the additional benefit of embedded DICOM technology. Through system integration, the use of energy efficient equipment, standard printer consumables and a three year on-site warranty, the C711 DM offers considerable savings for your day-to-day printing needs.

See it clearer with OKI

Talking directly with your medical imaging equipment, the C711 DM presents many opportunities for printing high quality, non-diagnostic images in HD mono and colour on a wide range of media. The significantly reduced costs of printing using the C711 DM, compared with traditional DICOM printing technologies, allows medical establishments to change their approach to printing.

Imagine the possibilities; print MRI scans in colour, offer mothers-to-be A4 ultrasound scans or keep sharp and clear non-diagnostic images in patients' records, and all at a very low cost. In fact, the same cost of printing as standard office documents.

Benefits at a Glance

- Fully integrated DICOM print server
- Up to 3 simultaneous imaging equipment connections
- Recommended usage - Nuclear medicine, ultrasound, MRI/CT scans, x-rays
- High quality non-diagnostic mono and colour printing of medical images
- No additional external hardware or software required
- Handles media from A6 to A4 wide 1200mm banners, from 64 to 250gsm
- Three year on-site warranty (product to be registered within 30 days of purchase)



C711 DM - Colour and Mono DICOM Embedded Printer

OVERVIEW

Description	The C711 DM is our most capable A4 DICOM imager. With high yield consumables, it can serve different modalities as X-ray, Ultrasound, PX
Imaging Technology	CMYK toner based engine with tandem technology. Light sources are LED arrays @ 600 dpi with multi level capability
Network	10/100 Base TX (RJ45) interface

GENERAL FEATURES

Print Format - Paper weight	A4 - 64 to 250 gsm
Dimensions (HxWxD)	389 x 435 x 546mm
Weight	28kg

DICOM SERVER FEATURES

Capability	DICOM 3.0 Print SCP
Maximum number of AET	25
Max # calling modalities	No limitations
Maximum connections	3 simultaneous
Dicom Server interface	Integrated web server
Log	Extended log capability in case of troubleshooting
LUT support	identity, reverse, lin-od
Image generation	Specific color and mono rendering optimized for medical images
Graphical Header	Capability to add a graphical header on each printout

AET DEPENDENT FEATURES

AET Name	Print SCP Application Entity Title (AET)
Printer Tray	Use preferred paper for your medical prints
Copy	Send one, print what you need
Border Density	Value from Black, White or Client
Empty image Density	Value from Black, White or Client
Replace Header by Logo	Use a graphical header instead of text for enhanced communication on the printouts
Header	Enter text that will print on top of page or select Logo
Footer	Edit the text on page bottom
Contrast	Add or remove contrast on the printouts
Brightness	Add or remove brightness on the printouts
Hi Quality Glossy Black	Print your greyscale images using best composite rendering
Greyscale, Cyan, Magenta and Yellow Gamma	Range from 0.01 to 2.90 by step of 0.01. Values lower than 1.00 will lighten, values higher than 1.00 will darken the image
Smoothing	Prompts C711 to use or not adjustable Sharpen filter to print
Polarity	Force image polarity inversion if needed

ACCESSORIES (ORDER NUMBERS)

2nd paper tray	44274502
3rd paper tray	44274502
Duplex	01272601
Cabinet	01219302

*Toner: A4 pages yield compliant with ISO/IEC 19798. Printer shipped with toner for 4,000 pages. Medical printing strongly alter these values.

**Drum: Average number of A4 pages printed.

Consumables Information: For the protection of your printer, and in order to ensure that you benefit from its full functionality, this model has been designed to operate only with genuine OKI toner cartridges. These can be identified by the OKI trademark. Any other toner cartridge may not operate at all, even if it is described as "compatible", and if it does work, your printer's performance and print quality may be degraded.



3 Year On-site Warranty

Our devices are manufactured to the highest standards of quality and technology, which has been confirmed by independent tests. We are so convinced of the high quality of our products that we are offering you an extension of the standard warranty period to 3 years at no extra cost to you. Simply register your product within 30 days of purchase to be eligible for our unique all inclusive offer. For further information please visit www.oki.co.uk/WARRANTY. Without registration, the standard pan-European 1 year warranty applies.

CONSUMABLES (ORDER NUMBERS)

Toner cartridges* (11,500 pages)	Cyan: 44318607; Magenta: 44318606; Yellow: 44318605
Toner cartridges* (11,000 pages)	Black: 44318608
Image drum** (20,000 pages)	Cyan: 44318507; Magenta: 44318506; Yellow: 44318505; Black: 44318508
Transfer Belt (60,000 pages)	44341902
Fuser Unit (60,000 pages)	44289103

OFFICE USAGE FEATURES

C711 DM inherits all features of standard product	Please refer to C711 specs sheet available on OKI websites
--	--

DICOM INTERFACE

On embedded web server	Full configuration of the server features, log viewing
-------------------------------	--

Print SCP AET Configuration

AET Name OKI

Activate Max Dicom Log ☐

Use US format paper ☐

Printer Tray AUTO

Copy CLIENT

Border Density CLIENT

Empty Image Density CLIENT

Replace Header by Logo ☐

Using Logo as Header reduce image printing area

Header

Footer DICOM Embedded Printer - Not for diagnostic use -
Editing this original statement footer is full responsibility of the user.

Contrast 0

Brightness 0

Hi Quality Glossy Black Mode ☐

Black gamma (0.01 to 2.90) 0.7

Cyan gamma (0.01 to 2.90) 0.85

Magenta gamma (0.01 to 2.90) 0.89

Yellow gamma (0.01 to 2.90) 0.82

Smoothing Filter CLIENT

Smoothing Level LOW

Polarity CLIENT




OKI EUROPE LTD.

Blays House, Wick Road
Egham, TW20 0HJ
United Kingdom
T +44(0) 20 8219 2190
F +44(0) 20 8219 2199

WWW.OKIEUROPE.COM

OKI

Open up your dreams

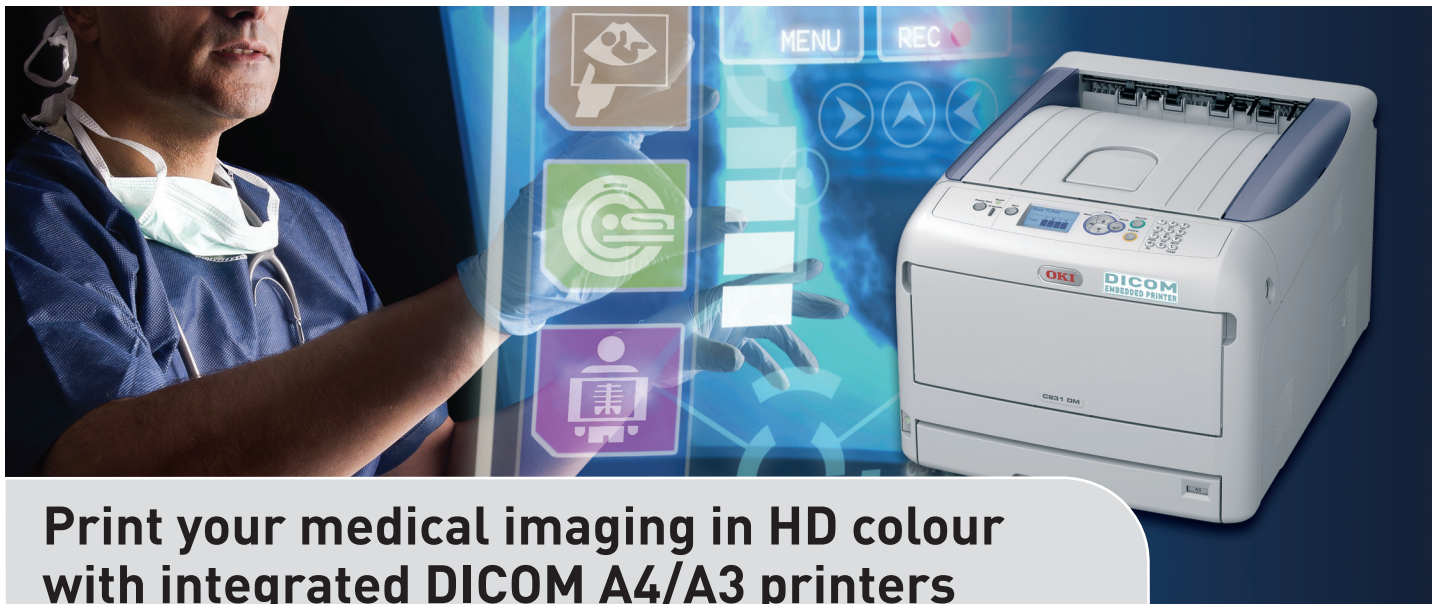

Print your medical imaging in HD colour with integrated DICOM A4/A3 printers

OKI is a leader in printing innovation, providing organisations with High Definition colour printing devices based on digital LED technology, and now our sophisticated range of printers integrate seamlessly with your clinical systems using DICOM.

A first in medical imaging and printing

This is a first in digital printer technology, combining the cost effectiveness and high quality output of an LED printer with embedded DICOM software, allowing you to print directly from medical equipment without the use of conversion software or external print servers. You no longer need to settle for expensive printing on specialist media and devices.

With the C831 DM you can now print your medical imaging cost effectively on standard media formats in vibrant, HD colour and in sizes up to A3. From nuclear medicine to scans (eg MRI, CT) and ultrasounds, these images can now be easily printed for your own non-diagnostic use or to provide a copy for your patients and clients to take away with them.

Reduce your costs and improve your printing

The C831 DM is not only perfect for your medical imaging printing, it can also be used for your standard office printing, from patient letters and forms to medical reports, allowing you to choose one machine that answers all the needs of your organisation, cutting down on the number of devices and consumables used.

The C831 DM is based on our existing, award winning printers, offering the same reliability, ease of use, low total cost of ownership, media flexibility and high quality output, with the additional benefit of embedded DICOM technology. Through system integration, the use of energy efficient equipment, standard printer consumables and a three year on-site warranty, the C831 DM offers considerable savings for your day-to-day printing needs.

See it clearer with OKI

Talking directly with your medical imaging equipment, the C831 DM presents many opportunities for printing high quality, non-diagnostic images in HD mono and colour on a wide range of media. The significantly reduced costs of printing using the C831 DM, compared with traditional DICOM printing technologies, allows medical establishments to change their approach to printing.

Imagine the possibilities; print MRI scans in colour, offer mothers-to-be A4 or A3 ultrasound scans or keep sharp and clear non-diagnostic images in patients' records, and all at a very low cost. In fact, the same cost of printing as standard office documents.

Benefits at a Glance

- Fully integrated DICOM print server
- Up to 3 simultaneous imaging equipment connections
- Recommended usage - Nuclear medicine, ultrasound, MRI/CT scans
- High quality non-diagnostic mono and colour printing of medical images
- No additional external hardware or software required
- Handles media from A6 to A3 wide 1200mm banners, from 64 to 256gsm
- Three year on-site warranty (product to be registered within 30 days of purchase)



C831 DM - Colour and Mono DICOM Embedded Printer

OVERVIEW	
Description	The C831 DM is a very small footprint A4/A3 DICOM imager. Perfect for nuclear medicine, ultrasound, CT, MRI
Imaging Technology	CMYK toner based engine with tandem technology. Light sources are LED arrays @ 600 dpi with multi level capability
Network	10/100 Base TX (RJ45) interface

GENERAL FEATURES	
Print Format - Paper weight	A4 and A3 - 64 to 256gsm
Dimensions (HxWxD)	360 x 449 x 552mm
Weight	37kg

DICOM SERVER FEATURES	
Capability	DICOM 3.0 Print SCP
Maximum number of AET	25
Max # calling modalities	No limitations
Maximum connections	3 simultaneous
Dicom Server interface	Integrated web server
Log	Extended log capability in case of troubleshooting
LUT support	identity, reverse, lin-od
Image generation	Specific color and mono rendering optimized for medical images
Graphical Header	Capability to add a graphical header on each printout

AET DEPENDENT FEATURES	
AET Name	Print SCP Application Entity Title (AET)
Printer Tray	Use preferred paper for your medical prints
Copy	Send one, print what you need
Page Format	Force the printout on the selected format
Border Density	Value from Black, White or Client
Empty image Density	Value from Black, White or Client
Replace Header By Logo	Use a graphical header instead of text for enhanced communication on the printouts
Header	Enter text that will print on top of page if Logo isn't used
Footer	Edit the text on page bottom
Contrast	Add or remove contrast on the printouts
Brightness	Add or remove brightness on the printouts
Hi Quality Glossy Black	Print your greyscale images using best composite rendering
Cyan, Magenta and Yellow Gamma	Range from 0.01 to 2.90 by step of 0.01. Values lower than 1.00 will lighten, values higher than 1.00 will darken the image
Smoothing	Prompts C831 to use or not adjustable Sharpen filter to print
Polarity	Force image polarity inversion if needed

ACCESSORIES (ORDER NUMBERS)	
Duplex	44717603
2nd/3rd/4th paper tray	44713903
Cabinet	01321101

CONSUMABLES (ORDER NUMBERS)	
Toner cartridges* (10,000 pages)	Cyan: 44844507; Magenta: 44844506; Yellow: 44844505; Black: 44844508
Image drum** (30,000 pages)	Cyan: 44844407; Magenta: 44844406; Yellow: 44844405; Black: 44844408
Transfer Belt (80,000 pages)	44846204
Fuser Unit (100,000 pages)	44848805

OFFICE USAGE FEATURES	
C831 DM inherits all features of standard product	Please refer to C831 specs sheet available on OKI websites

DICOM INTERFACE	
On embedded web server	Full configuration of the server features, log viewing

Print SCP AET Configuration

AET Name OKI

Activate Max Dicom Log

Use US format paper

Printer Tray AUTO

Copy CLIENT

Page Format CLIENT

Border Density CLIENT

Empty Image Density CLIENT

Replace Header by Logo

Using Logo as Header reduce image printing area

Header

Footer DICOM Embedded Printer - Not for diagnostic use -
Editing this original statement footer is full responsibility of the user

Contrast 0

Brightness 0

Hi Quality Glossy Black Mode

Cyan gamma (0.01 to 2.90) 0.85

Magenta gamma (0.01 to 2.90) 0.89

Yellow gamma (0.01 to 2.90) 0.82

Smoothing Filter CLIENT

Smoothing Level LOW

Polarity CLIENT



*Toners: A4 pages yield compliant with ISO/ISC 19798. Printer shipped with toner for 2,500 A4 pages yield compliant with ISO/ISC 19798. Medical printing strongly alter these values.
**Drum: Average number of A4 pages printed.

Consumables Information: For the protection of your printer, and in order to ensure that you benefit from its full functionality, this model has been designed to operate only with genuine OKI toner cartridges. These can be identified by the OKI trademark. Any other toner cartridge may not operate at all, even if it is described as "compatible", and if it does work, your printer's performance and print quality may be degraded.



3 Year On-site Warranty


Our devices are manufactured to the highest standards of quality and technology, which has been confirmed by independent tests. We are so convinced of the high quality of our products that we are offering you an extension of the standard warranty period to 3 years at no extra cost to you. Simply register your product within 30 days of purchase to be eligible for our unique all inclusive offer. For further information please visit www.oki.co.uk/WARRANTY. Without registration, the standard pan-European 1 year warranty applies.

OKI EUROPE LTD.

Blays House, Wick Road
Egham, TW20 0HJ
United Kingdom
T +44(0) 20 8219 2190
F +44(0) 20 8219 2199
www.okieurope.com

OKI

Open up your dreams

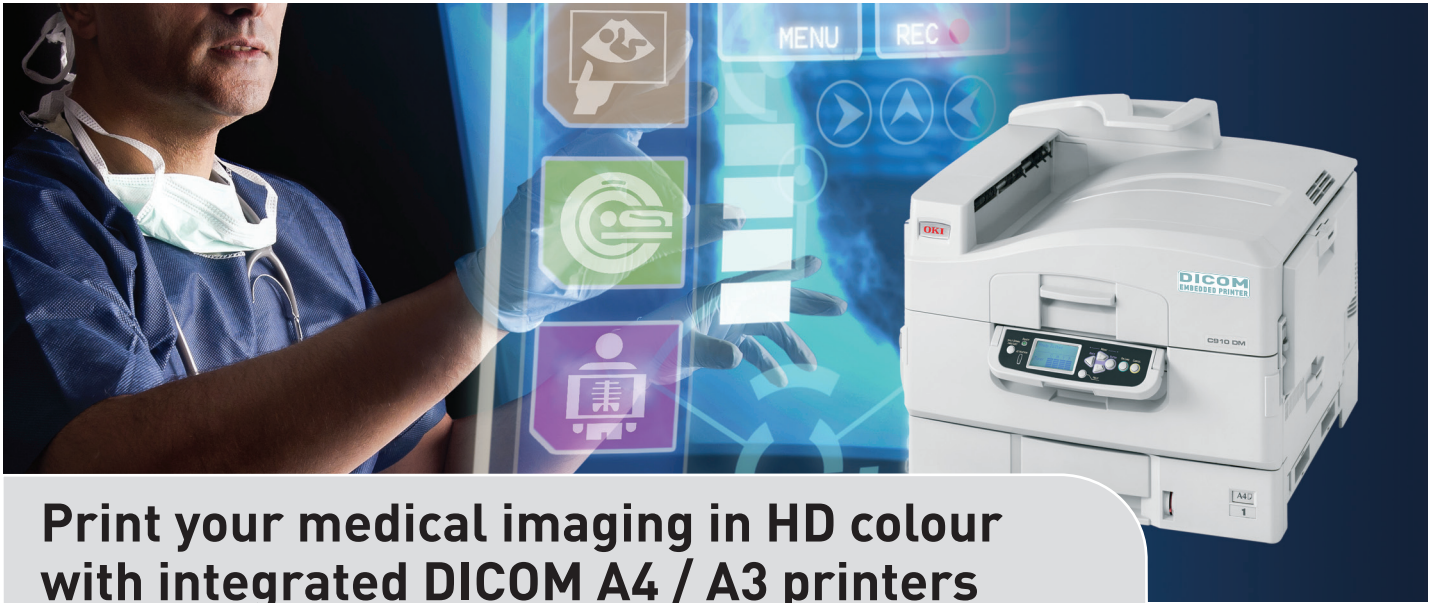

Print your medical imaging in HD colour with integrated DICOM A4 / A3 printers

OKI is a leader in printing innovation, providing organisations with High Definition colour printing devices based on digital LED technology, and now our sophisticated range of printers integrate seamlessly with your clinical systems using DICOM.

A first in medical imaging and printing

This is a first in digital printer technology, combining the cost effectiveness and high quality output of an LED printer with embedded DICOM software, allowing you to print directly from medical equipment without the use of conversion software or external print servers. You no longer need to settle for expensive printing on specialist media and devices.

With the C910 DM you can now print your medical imaging cost effectively on standard media formats in vibrant, HD colour and in sizes up to A3+. From x-rays (eg dental, veterinary, hospital, clinic) to nuclear medicine, scans (eg MRI, CT) and ultrasounds, these images can now be easily printed for your own non-diagnostic use or to provide a copy for your patients and clients to take away with them.

Reduce your costs and improve your printing

The C910 DM is not only perfect for your medical imaging printing, it can also be used for your standard office printing, from patient letters and forms to medical reports, allowing you to choose one machine that answers all the needs of your organisation, cutting down on the number of devices and consumables used.

The C910 DM is based on our existing, award winning printers, offering the same reliability, ease of use, low total cost of ownership, media flexibility and high quality output, with the additional benefit of embedded DICOM technology. Through system integration, the use of energy efficient equipment, standard printer consumables and a three year on-site warranty, the C910 DM offers considerable savings for your day-to-day printing needs.

See it clearer with OKI

Talking directly with your medical imaging equipment, the C910 DM presents many opportunities for printing high quality, non-diagnostic images in HD mono and colour on a wide range of media. The significantly reduced costs of printing using the C910 DM, compared with traditional DICOM printing technologies, allows medical establishments to change their approach to printing.

Imagine the possibilities; print MRI scans in colour, offer mothers-to-be A4 or A3 ultrasound scans or keep sharp and clear non-diagnostic images in patients' records, and all at a very low cost. In fact, the same cost of printing as standard office documents.

Benefits at a Glance

- Fully integrated DICOM print server
- Up to 5 simultaneous imaging equipment connections
- Recommended usage - Nuclear medicine, ultrasound, MRI/CT scans, x-rays
- High quality non-diagnostic mono and colour printing of medical images
- No additional external hardware or software required
- Handles media from A6 to A3+ and A3 wide 1200mm banners, from 64 to 300gsm
- Three year on-site warranty (product to be registered within 30 days of purchase)



C910 DM - Colour and Mono DICOM Embedded Printer

OVERVIEW	
Description	Initially designed for graphic application, the C910 DM has been heavily optimized for DICOM reproduction needs, resulting in very high quality output for all medical imaging centers needs.
Imaging Technology	CMYK toner based engine with tandem technology. Light sources are LED arrays @ 600 dpi with multi level capability
Network	10/100/1000 Base TX (RJ45) interface

GENERAL FEATURES	
Print Format - Paper weight	A4 and A3 - 64 to 300gsm
Dimensions (HxWxD)	471 x 654.5 x 623mm
Weight	76kg

DICOM SERVER FEATURES	
Capability	DICOM 3.0 Print SCP
Maximum number of AET	25
Max # calling modalities	No limitations
Maximum connections	5 simultaneous
Dicom Server interface	Integrated web server
Log	Extended log capability in case of troubleshooting
LUT support	identity, reverse, lin-od
Image generation	Specific color and mono rendering optimized for medical images
Graphical Header	Capability to add a graphical header on each printout

AET DEPENDENT FEATURES	
AET Name	Print SCP Application Entity Title (AET)
Printer Tray	Use preferred paper for your medical prints
Copy	Send one, print what you need
Page Format	Force the printout on the selected format
Border Density	Value from Black, White or Client
Empty image Density	Value from Black, White or Client
Replace Header By Logo	Use a graphical header instead of text for enhanced communication on the printouts
Header	Enter text that will print on top of page if Logo isn't used
Footer	Edit the text on page bottom
Contrast	Add or remove contrast on the printouts
Brightness	Add or remove brightness on the printouts
Hi Quality Glossy Black	Print your greyscale images using best composite rendering
Greyscale, Cyan, Magenta and Yellow Gamma	Range from 0.01 to 2.90 by step of 0.01. Values lower than 1.00 will lighten, values higher than 1.00 will darken the image
Activate Halftoning	For advanced user only, allow printer screening selection
Smoothing	Prompts C910 to use or not adjustable Sharpen filter to print
Polarity	Force image polarity inversion if needed

ACCESSORIES (ORDER NUMBERS)	
2nd/3rd paper tray	42831303
Duplex	42797203
High Capacity Feeder	42831503
Cabinet	09004256

*Toner: Number of A4 pages @ 5% coverage. Printer shipped with toner for 7,500 pages @ 5%. Medical printing strongly alter these values.
 **Drum: Average number of A4 pages printed.

Consumables Information: For the protection of your printer, and in order to ensure that you benefit from its full functionality, this model has been designed to operate only with genuine OKI toner cartridges. These can be identified by the OKI trademark. Any other toner cartridge may not operate at all, even if it is described as "compatible", and if it does work, your printer's performance and print quality may be degraded.



3 Year On-site Warranty

Our devices are manufactured to the highest standards of quality and technology, which has been confirmed by independent tests. We are so convinced of the high quality of our products that we are offering you an extension of the standard warranty period to 3 years at no extra cost to you. Simply register your product within 30 days of purchase to be eligible for our unique all inclusive offer. For further information please visit www.oki.co.uk/WARRANTY. Without registration, the standard pan-European 1 year warranty applies.

CONSUMABLES (ORDER NUMBERS)	
Toner cartridges* (15,000 pages)	Cyan: 44036023; Magenta: 44036022; Yellow: 44036021; Black: 44036024
Image drum** (20,000 pages)	Cyan: 44035519; Magenta: 44035518; Yellow: 44035517; Black: 44035520
Transfer Belt (100,000 pages)	42931603
Fuser Unit (100,000 pages)	42931703
Waste Toner Box (30,000 pages)	42869403
Finisher Staples (15,000 staples)	42937603

OFFICE USAGE FEATURES	
C910 DM inherits all features of standard product	Please refer to C910 specsheet available on OKI websites

DICOM INTERFACE	
On embedded web server	Full configuration of the server features, log viewing

Print SCP AET Configuration

AET Name

OKI

Activate Max Dicom Log

☐

Use US format paper

☐

Printer Tray

AUTO

Copy

CLIENT

Page Format

CLIENT

Border Density

CLIENT

Empty Image Density

CLIENT

Replace Header by Logo

☐

Using Logo as Header reduce image printing area

Header

Footer

DICOM Embedded Printer - Not for diagnostic use -
Editing this original statement footer is full responsibility of the user

Contrast

0

Brightness

0

Hi Quality Glossy Black Mode

☐

Black gamma (0.01 to 2.90)

1.0

Cyan gamma (0.01 to 2.90)

1.0

Magenta gamma (0.01 to 2.90)

1.0

Yellow gamma (0.01 to 2.90)

1.0

Activate Halftoning

☐

Smoothing Filter

CLIENT

Smoothing Level

LOW



OKI EUROPE LTD.

Blays House, Wick Road
 Egham, TW20 0HJ
 United Kingdom
 T +44(0) 20 8219 2190
 F +44(0) 20 8219 2199
www.okieurope.com

OKI

Open up your dreams